INSTRUCTIONS FOR RE-USE OF:
All GMD Reusable Trocars

LANGUAGES

English.....	1	German	11
Danish	3	Italian	13
Dutch	5	Norwegian	15
French	7	Portuguese	17
Spanish	9	Swedish	19

English

INSTRUCTIONS FOR RE-USE OF:

GMD Reusable Trocar Models

Rx ONLY- Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

1. Introduction

This instruction for use (IFU) provides important information, which will enable you to use the GMD Reusable Trocars in a safe and effective manner. These instructions will also help you to avoid potential application hazards to the patient. These instructions are intended for use by persons with the required specialist knowledge and training. This IFU is therefore to be stored in such a place that is accessible to the user at all times.

2. General Safety Precautions

The manufacturer will not accept any claims for liability or offer a guarantee for damage, which is attributed to the improper repair or modifications made to the GMD Reusable Trocars by unauthorized persons.

The user of this product is solely responsible for any damage resulting from:

- A. Improper handling
- B. Incorrect maintenance
- C. Non observance of the contents of this IFU.

3. Symbols and Definitions

The following symbols are used in conjunction with this device:

	Batch code
	Consult Instructions for Use
	This Product Contains No Detectable Latex
	Product Number
	No DEHP

4. Warnings

- Follow instructions and warnings as issued by manufacturers of any decontaminants, disinfectants and cleaning agents used. Wherever possible avoid use of the mineral acids and harsh, abrasive agents.
- No part of the process shall exceed 140°C.
- The trocar aluminum handle is damaged by high alkaline solutions (pH>10).
- Care must be taken when handling the trocar device to avoid dropping or rough contact with other devices.
- Do not use metallic or abrasive cleaning agents.
- Do not permit sharp instruments or edges to contact the trocar devices.

Note: when reprocessing GMD trocars, always handle with care, wearing protective clothing, gloves and eyewear in accordance with local Health & Safety procedures.

5. Device Description

All GMD Reusable Trocars are supplied as non-sterile reusable devices and are considered accessories to various models of the GMD Universal Sling™ for the treatment of female stress urinary incontinence (SUI).

6. Intended Use

The GMD Reusable Trocars should only be used for the placement of GMD Universal Slings™ pursuant to each GMD Universal Sling™ model's Instructions for Use. The GMD Universal Slings are intended for use in women as a suburethral sling for the treatment of stress urinary incontinence (SUI) resulting from either urethral hypermobility and/or intrinsic sphincter deficiency.

7. Instrument Care

A. Limitations on Reprocessing

- GMD reusable trocars have been successfully tested to 50 reprocessing cycles with autoclave.
- End of life is normally determined by wear and damage after each use.

B. Cleaning and Sterilization Instructions

Decontamination and sterilization prior to use are the responsibility of the hospital facility as the GMD Reusable Trocars are provided as nonsterile.

C. From Point Of Use

- Wherever possible, do not allow blood, debris or bodily fluids to dry on the trocar devices. For best results, and to prolong the life of the trocar device, reprocess immediately after use. If they cannot be reprocessed immediately, use an enzymatic foam spray cleaner to help prevent soil from drying.
- Universal precautions for handling contaminated/biohazardous materials should be observed.
- Instruments should be cleaned within 30 minutes of use to minimize the potential for drying prior to cleaning.

D. Cleaning – Automated

- Use only either CE marked or validated washer-disinfector machines and low foaming, non-ionizing cleaning agents and detergents following the manufacturer's instructions for use, warnings and recommended cycles.

- Prior to placing the test devices in the washer, rinse the devices under running tap water to remove soil.
- Load trocar devices carefully avoiding contact with other devices.
- Place heavy devices with care in the bottom of containers, taking care not to overload wash baskets.
- Ensure that soft, high purity water which is controlled for bacterial endotoxins is used in the final rinse stage.
- The cleaning cycle should be greater or equivalent to:

PHASE	RECIRCULATION TIME (MINUTES)	WATER TEMPERATURE	DETERGENT TYPE AND CONCENTRATION
Pre-wash 1	02:00	Cold tap water	N/A
Enzyme Wash	01:00	Hot tap water	Enzol [®] , 1 oz/gallon or equivalent
Wash 1	02:00	60°C (Set Point)	Renu-Klenz [®] , ¼ oz/gallon or equivalent
Rinse 1	01:00	Heated 60.0°C	N/A

- After Rinse 1 phase, stop the cycle and remove the devices from the washer.
- Dry the devices using a clean cloth. Pressurized air can be used to dry the device if applicable. Do not exceed 40 psi.

Note: After manually cleaning, pass all devices through an automated cleaning cycle to achieve disinfection.

Note: These instructions have been validated using a washer-disinfector cycle validated to include two cold rinses at <35°C, a detergent cycle and a rinse cycle both at >50°C, a disinfection cycle operating at a temperature of between 80°C and 87°C for a minimum holding time of 1 minute (actual holding time in excess of 2 minutes 50 seconds) and a 20 minute drying cycle. The detergent used was a low-foaming, non-ionising spray wash detergent cleaner (max 12pH) and the rinse aid a neutral pH low-foaming, non-ionic surfactant with isopropyl alcohol.

E. Cleaning – Manual

- **Manual cleaning is not advised if an automatic washer-disinfector is available. If this equipment is not available, use the following process:**
1. Using a double sink system (wash / rinse) dedicated for device cleaning (not used for hand washing). Ensure that the water temperature does not exceed 35°C.
 2. Promptly rinse each device under cold running tap water to remove all visible soil. Fully immerse the devices in an enzymatic detergent, Enzol[®], according to manufacturer's recommendations using lukewarm tap water for a minimum of two minutes. Scrub the devices using a soft bristled brush paying particular attention to hard-to-reach locations. Rinse the devices with reverse osmosis/deionized (RO/DI) water for a minimum of one minute to remove any residual detergent. Dry the devices using a clean soft cloth. If visible soil remains, repeat cleaning procedure again.

Note: manual cleaning is NOT a disinfection process. When manual cleaning is used it may not be possible to disinfect the device prior to further handling.

F. Cleaning – Inspection

- After cleaning, visually inspect all surfaces including the trocar handle, needle and needle T-slot for complete removal of soil and fluids. If ANY soil or fluid is still visible, return the device for repeat cleaning/decontamination.

G. Inspection and Function Testing

- Visually inspect and check:
 - o All trocar devices handles, needles and needle T-slots for damage and wear.
 - o Trocar needle is firmly attached to the trocar device handle
- Remove for replacement any pitted, worn out (loose needle/handle), flaking, fractured or damaged devices.

H. Sterilization

- Follow **ANSI/AAMI, Comprehensive guide to steam sterilization and sterility assurance in health care facilities**, and sterilizer manufacturer's written instructions for cycle parameters (exposure time, temperature, and dry time).

I. Storage

- Ensure trocar devices are dry before storage, and stored in dry, clean conditions at an ambient room temperature.

Note: It is the responsibility of the re-processor to ensure that the reprocessing as actually performed using equipment, materials and personnel in the reprocessing facility achieve the desired result. This requires validation and routine monitoring of the process. Likewise any deviation by the re-processor from the instructions provided must be properly evaluated for effectiveness and potential adverse consequences.

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GMD, Inc.